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CORRECTIVE MAINTENANCE / MANUTENCIÓN CORRECTIVA

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All recommendations for use are found in this Manual, but if any problems or malfunctions are detected please contact the Evotech Technical Support Center to correct the problem. Do not attempt to open the product and / or repair it yourself or with help of persons not trained / authorized to do so, this may aggravate the problem or even generate faults that could compromise product safety.

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The High Speed was designed to have a maximum noise level of 80 dB, as long as it is installed, kept clean and maintained, transported and operated according to these use instructions.

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For warranty's information see the certificate of the product. The warranty's product covers manufacture defects counted from the date of sale to the first client or the delivery of the warranty certificate. The warranty becomes invalid in case any of the following situations is verified:

- **Installation and/or operation in disagreement with this manual;**
- **Fall;**
- **Damages to the external finishing and/ or accessories whose the manufacturer has not given any cause;**
- **Without lubrication.**

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With the purpose of offering a service that guarantees 100% satisfaction, we present to the customer the Evotech Technical Support Center, an innovation in the service that came to provide agility, safety and transparency in the maintenance process of your Evotech product. We have an infrastructure and a qualified professionals' team, guaranteeing the most complete technical assistance for the entire Evotech product line.

- **Trained technicians and engineers;**
- **Last generation machinery and equipment;**
- **Original components.**

To use the service offered by the Evotech Technical Support Center, please, contact us: E-mail: cbanhos@grupoevotech.com.br / Phone: + 55 (16) 3951-7542

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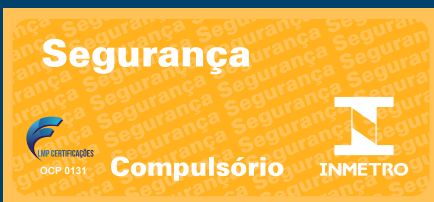
The High Speed is one dental unit accessory. It is easy to handle this equipment, it may be coupled in any kind of unit, by following all the products directions for its adequate use.

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The equipment is used in dental treatments and for the confection of dental prothesis.

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The High Speed Evotech is designated for dental procedures and can only be used and installed by a qualified and capacitated professional on dental area.

Its use for any other purpose is not allowed. Handling it improperly constitutes a danger source.

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The equipment can be disinfected with alcohol 70° GL, applied on its surface with a cloth or spray.

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The equipment must be sterilized before and after each treatment. Pack the product in packaging for sterilization and dispose only in an autoclave at a maximum temperature 135° C and 2.2 Bar.

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Keep in a place protected from the sun, heat, dust and humidity; Keep away from chemicals and cleaning agents; Carry out the transport only in its respective packaging; Ambient temperature (transport and storage): -10° to 55°C; Relative air humidity (transport and storage): 0% to 85%; Atmospheric pressure: 70kPa to 106kPa; Maximum stacking: 8 units.

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1 - Fit the high speed to the unit's terminal observing the ducts direction.

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2 - Screw the unit's terminal extremity to get fixed to the equipment.

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The dental unit's terminal must be manufactured in accordance with ISO 9168 norm.

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1 - Fit the bur manually into the clamp and press against the metallic support of the bur extractor. (Fig. 3).

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2 - To remove the bur, insert the equipment head on latch pin side, and then hold the bur and press the head on the extractor (Fig. 4).

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1 - Press the central part of the activator button, located on the superior part of the high speed head to open the clamp in order to enable the bur's fixing (figure 5) and removal (figure 6).

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Use only burs according to norm ISO 1797-1 type 3 with the following dimensions:
a - diameter between 1.59 and 1.60mm
b - minimum recommended length of the rotary instrument shaft is 12mm.
c - total maximum recommended length of the rotary tool is 22mm.

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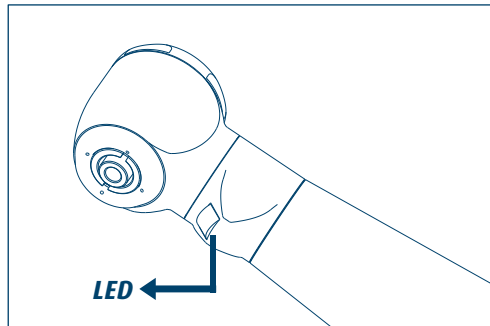
The luminosity of the LED can vary according to the working pressure and rotation of the turbine.

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1 - Lubricate before each sterilization cycle.

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2 - To lubricate, keep the oil bottle upright, insert your applicator at the high speed entry and inject oil for 2 seconds.

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3 - After lubrication the high speed must be autoclaved.

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The Evotech lubricant spray or other specific lubricant for dental handpieces should be used.

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- Do not expose to temperature above 55°C.
- Keep the bottle far from spark or fire.
- Do not pierce or burn the bottle neither if it's empty.
- Avoid contact to the eyes.

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Like any other electromedical device, High Speed is composed of several materials, such as metals, rubbers, etc. Therefore, in order to avoid environmental contamination or the improper use of the equipment after its destruction, it must be disposed of in an appropriate place (following local legislation for waste disposal conditions).

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Evotech: It declares that the materials used in its Applied Parts (according to the NBR IEC 60601-1 standard) in High Speed have been widely used in the dental field over the years, guaranteeing its biocompatibility. In addition, the consumable items of the equipment are: lubricating oil, distilled water and drills.

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